


Dexcom CLARITY®

Get Clinically Relevant Insights in Minutes

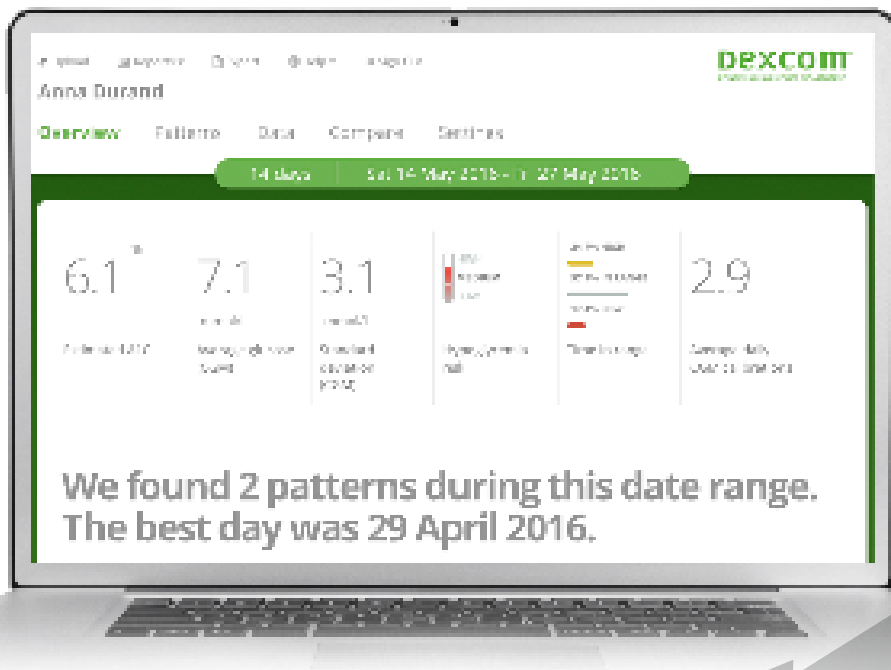

Dexcom
CLARITY®

Six Unique, Interactive Reports

Patients and healthcare professionals focus on the same issues when it comes to hyper- or hypoglycemia. Dexcom CLARITY brings these to the forefront - presenting the most relevant patterns and trends to help make better diabetes management decisions.

1

Nighttime
Lows

2

Daytime
Lows

3

Nighttime
Highs

4

Daytime
Highs

Contributing factors:

Contributing
Patterns



Rebound Lows
and Highs





Sustained Lows
and Highs

Overview Report

The Overview report presents up to four clinically relevant patterns, as well as the patient's Best Day. This quick summary can help focus the discussion on problem areas contributing to hyper- and hypoglycemia.

This report has two main parts:

- 1 Dashboard
- 2 Patterns overview

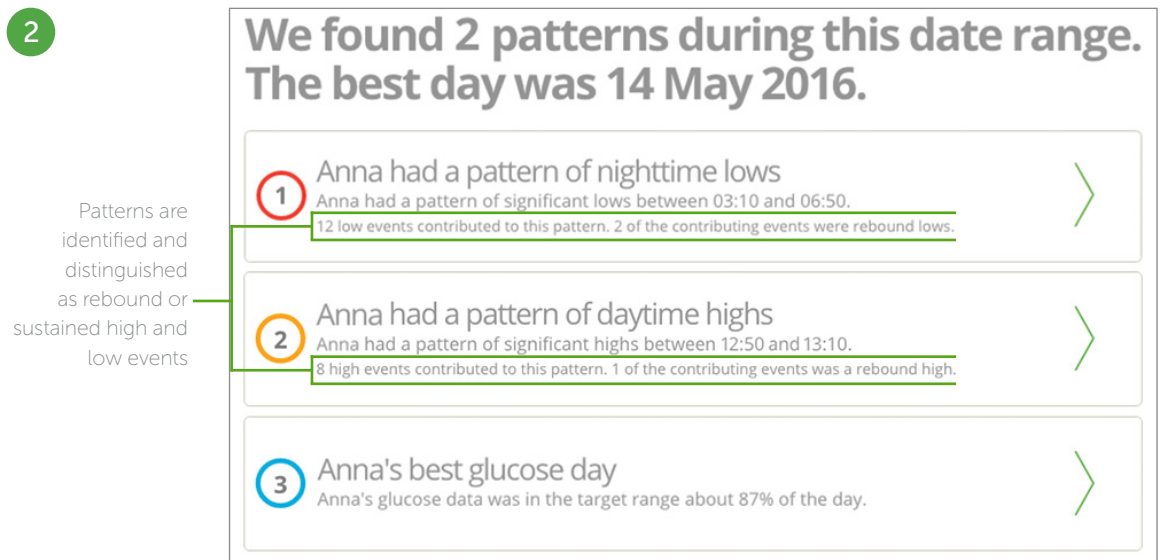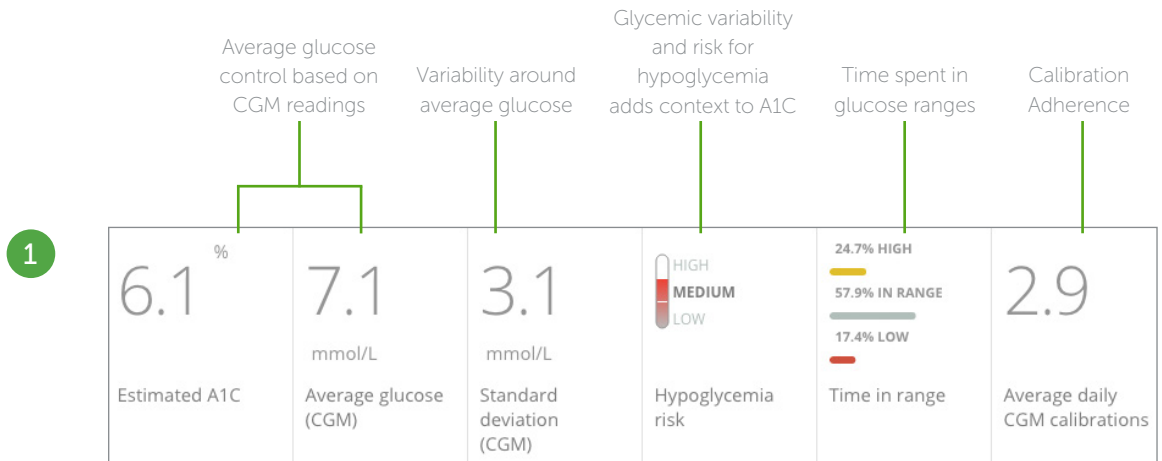


Each pattern highlights:

- Time of event
- Number of events
- Contributing events

Patterns Report


With the Patterns report, you can dive deeper into each of the four clinically relevant patterns. Each pattern is represented by a series of graphs of the days that contribute to that pattern. If a home user inputs an event into their CGM device, such as exercise, it is shown as an icon below the graph.

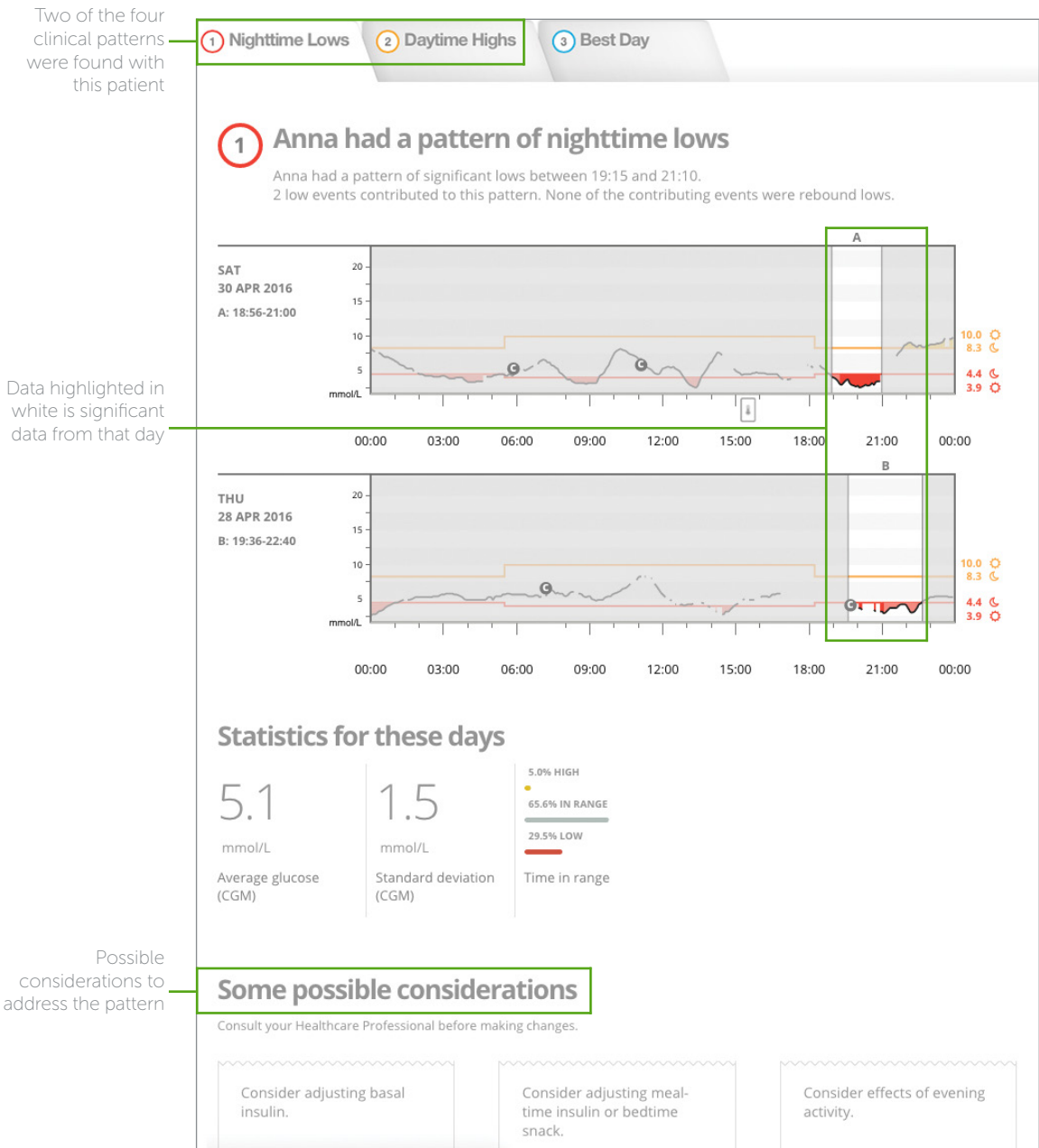

Data Report: Trends

The Trends view shows aggregate data for CGM glucose readings. In this report:

- Longer bars represent greater glycemic variability.
- Clinically significant hypoglycemia patterns are red - the most significant are bright red.
- Hyperglycemia patterns are yellow - the most significant are bright yellow.
- Outlier data is removed - top 25% and bottom 15% of data.

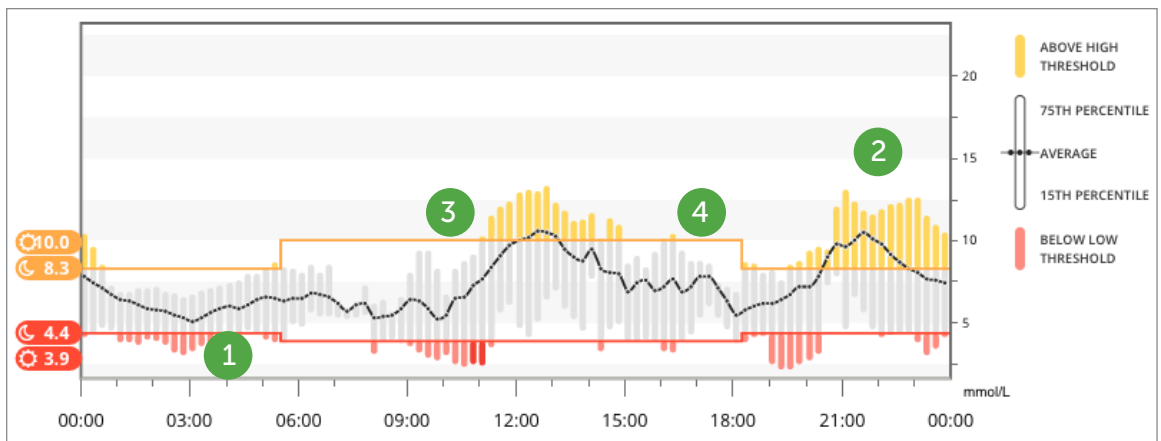

Glucose Pattern Management*


CLARITY may not identify a clinical pattern. The best practice is a stepwise approach that will help you identify patient challenges with hypo- and hyperglycemia.

- 1 **Hypoglycemia.** Determine when hypoglycemia occurs and prioritize hypoglycemia overnight (1st) and hypoglycemia day/night (2nd). In this example the patient is experiencing overnight hypoglycemia (1st) as well as hypoglycemia during the morning and evening hours (2nd).
- 2 **Overnight glucose control.** Determine if there is overnight hyperglycemia. Appropriate levels of basal insulin should keep glucose values in target range throughout the night. This is a problem in the above example.
- 3 **Pre-prandial glucose control.** Determine if there is pre-meal hyperglycemia. In this example patient's lunchtime was at 11:00. No pre-prandial hyperglycemia is detected.
- 4 **Post-prandial glucose control.** Determine if there is post-meal hyperglycemia. In this example a pronounced post-prandial hyperglycemia is after lunchtime.

* Use your professional judgement when interpreting CGM data

Data Report: Overlay

The Overlay view features CGM tracings to spot trends and compare data from different days. Each line on the graph represents one day's data over time. Each graph can contain up to 7 days of all sensor data from the selected date range.

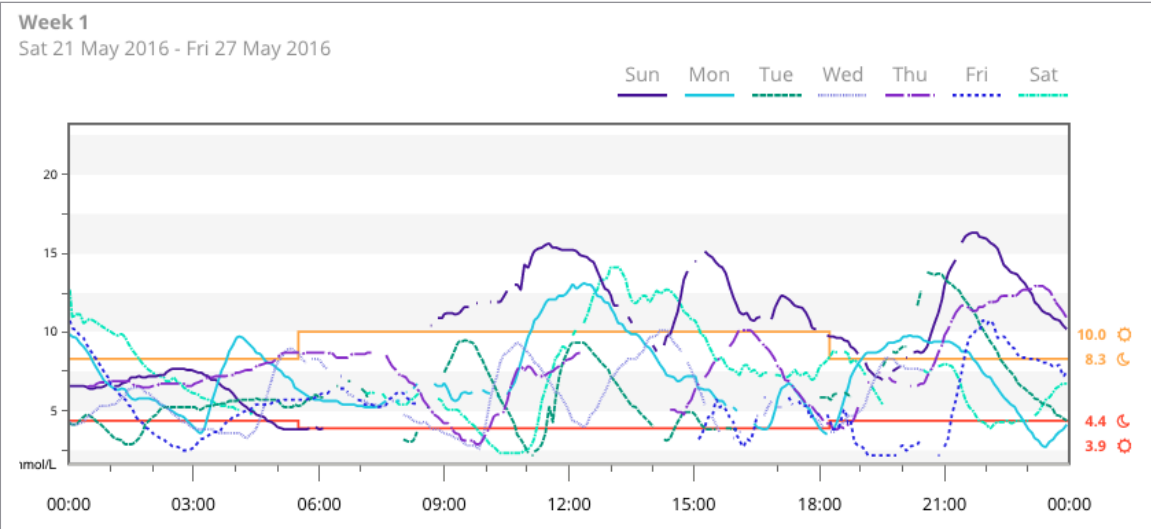

Customize reports by applying filters to the graphs.

View data from only certain days of the week

Choose between nighttime and day-time patterns

Compare different High and Low glucose events, rebound and sustained high and low events

View days when patient did not calibrate or CGM did not record data

DAYS	TIME OF DAY	EVENTS	USAGE
		<div><div>None</div><div>Lows</div><div>Highs</div><div>Sustained Lows</div><div>Sustained Highs</div><div>Rebound Lows</div></div>	

Data Report: Daily

The Daily view displays glucose data over time in daily graphs. The graphs are displayed with the most recent day first. Every glucose reading is displayed in this report.




Calibration Icon
Daily contributions to the pattern are in red and yellow




Compare Report

The Compare report encourages progress and highlights the challenges patients may be facing each visit.

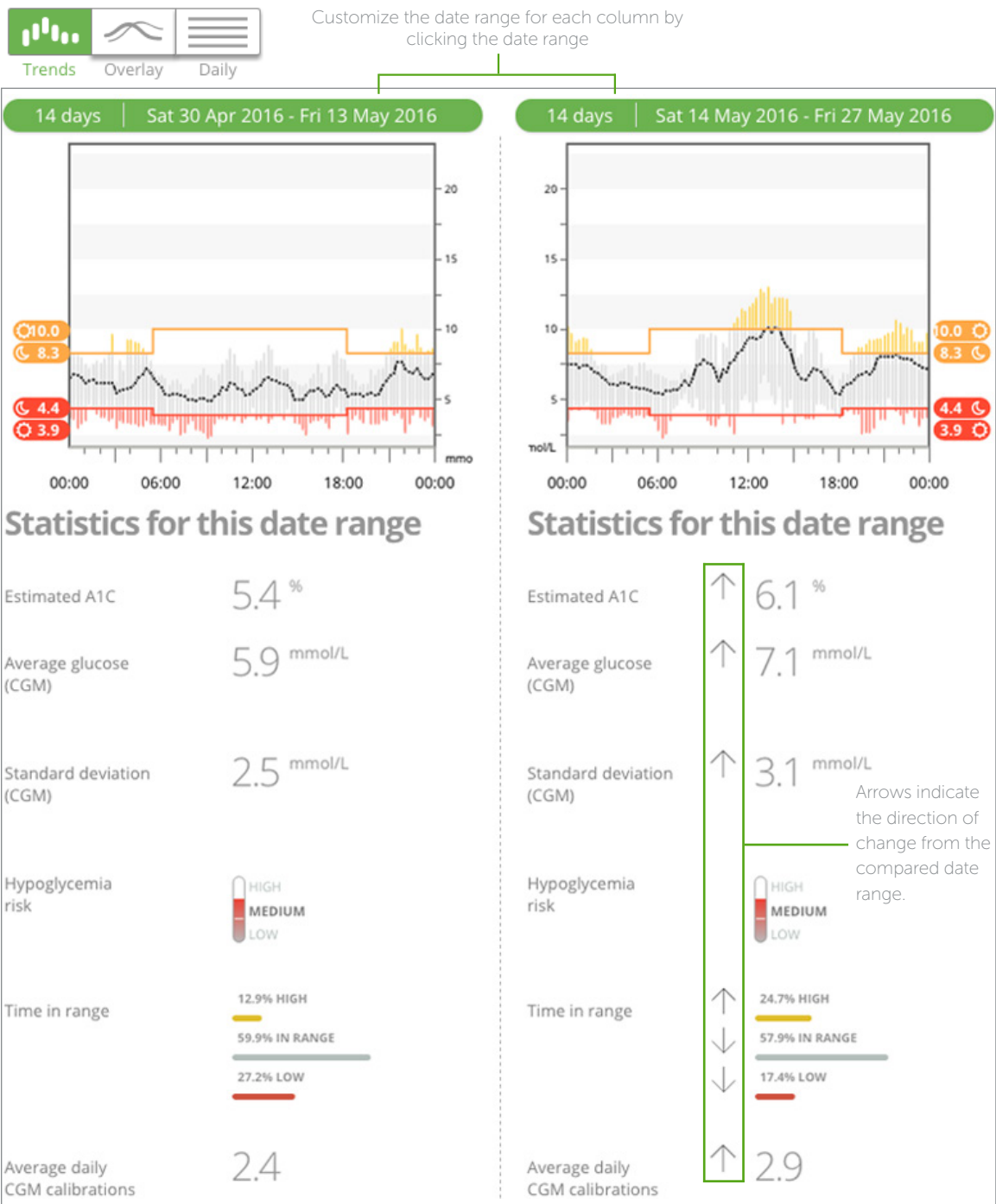

Save or Print Reports


The Overview report is just one of five reports you can save or print that can be easily referenced during each patient visit.

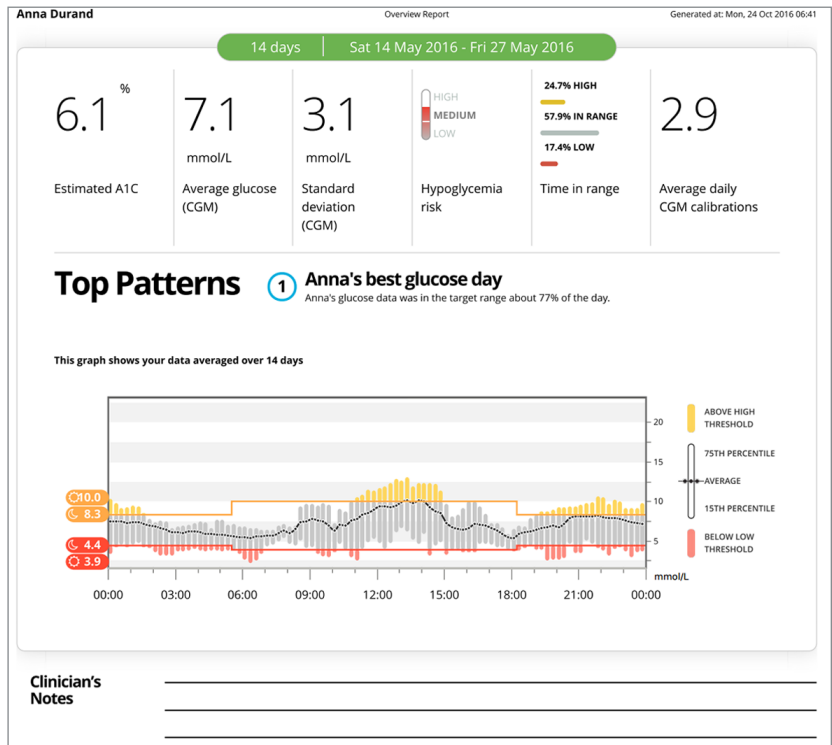

Settings

Use the Settings page to customize glucose time periods for target ranges. Changes you make to a patient's settings only apply at the clinic and do not change the patient's personal CLARITY account or any CGM settings.

Reports are generated based on these settings.

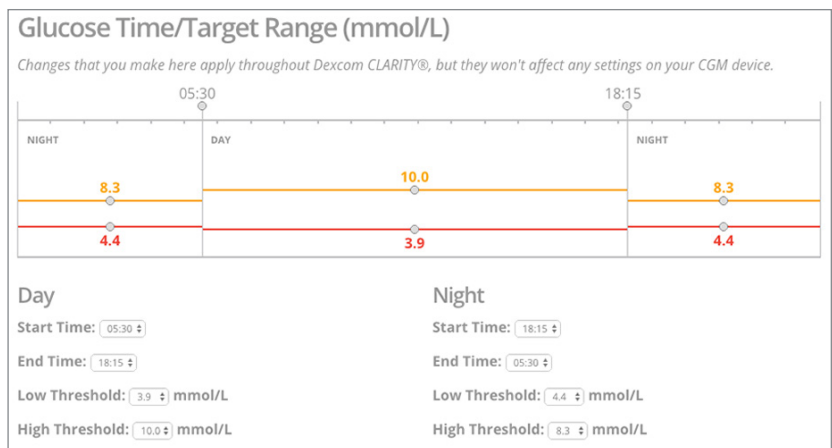

Dexcom CLARITY is not yet available for all countries.
See www.clarity.dexcom.eu for details.

BRIEF SAFETY STATEMENT

The web-based Dexcom CLARITY® software is intended for use by both home users and healthcare professionals to assist people with diabetes in the review, analysis, and evaluation of historical CGM data to support effective diabetes management. It is intended for use as an accessory to CGM devices with data interface capabilities. The software should not be relied on for medical advice. Home users must consult a healthcare professional before making any medical interpretation or therapy adjustments from the information. Healthcare professionals should use information from the software in conjunction with other clinical information available to them.

Dexcom
CLARITY®