

NEW CareLink™ Personal Uploader

Reliable connectivity for ease of data upload

<https://carelink.minimed.eu>*

INSTALL GUIDE

made easier to partner with your Health Care Professional

FOR TECHNICAL QUESTIONS:

Please contact our HelpLine


at 1-800-284-4416 or

diabetesps.canada@medtronic.com

IMPROVE YOUR HEALTH CARE PROFESSIONAL VISITS BY UPLOADING TO CARELINK™



Understand your glucose pattern

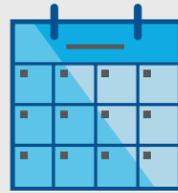


Upload regularly



Collaborate with your Health Care Professional

GET STARTED BY DOWNLOADING THE NEW UPLOADER INSTALL FILE


A one-time download per computer where you wish to upload your device.

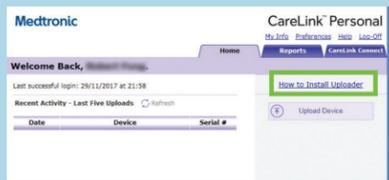

- 1 Log in to CareLink™ Personal
 - Click "How to install Uploader"



- 2 Click the download link and select a location to save the uploader file. (ex.: your desktop)

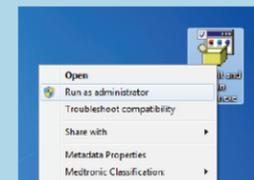


- 3 Find the downloaded file right click and select "Run as administrator"

INSTALL THE UPLOADER




- 1 Choose your language

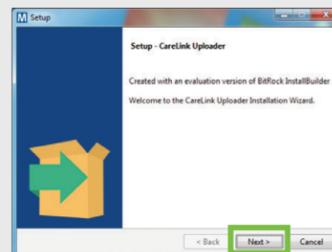


- 2 Click "Next"



- 3 Click "Next"

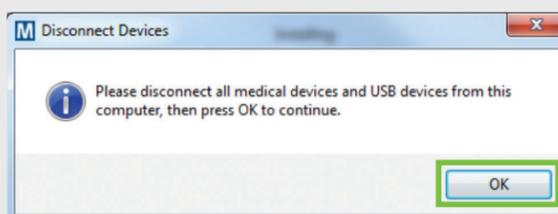

- 4 Make sure your CareLink™ USB or meter are not plugged in and click "OK"

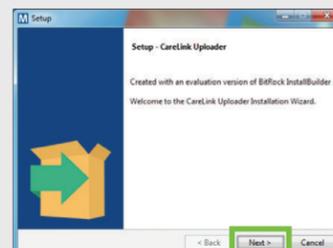

- 5 Click "Finish" to complete the installation



NOW, YOU'RE READY TO REGISTER AND UPLOAD

The uploader is installed and will run in the background, no need to launch it.

Return to your CareLink™ Personal Homepage as you are ready to register and upload your Pump or Meter to Carelink™.

Please turn over for upload instructions.

Medtronic

NEW CareLink™ Personal Uploader


Reliable connectivity for ease of data upload

UPLOAD GUIDE

Your first step to understanding more about your diabetes

FOR TECHNICAL QUESTIONS:
Please contact our HelpLine
at 1-800-284-4416

INITIAL FOUR-STEP PROCESS TO ADD YOUR PUMP OR METER

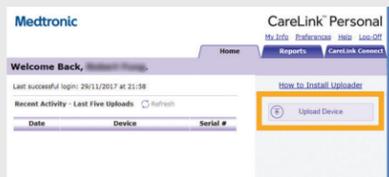

With your Internet Explorer, Chrome or Firefox web browser, navigate to the CareLink web page (<https://carelink.minimed.eu>*)



1 Choose pump




2 You can upload your 630G, Paradigm and all pump models



3

- Enter pump serial number.
- Enter all numbers and letters for 630G and just numbers for the Paradigm.

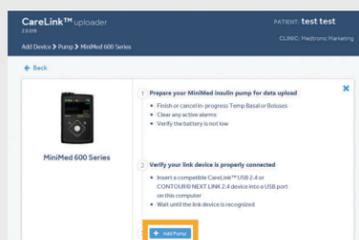

4

- Make sure the pump is ready
- Click "Add Pump"

TO ENSURE SUCCESS, GET EVERYTHING READY Plug your CareLink USB or meter into your computer or Contour Next Link Wireless Blood Glucose Meter

- Place your pump as close as possible to your computer
- Keep the meter linked with the pump



Make sure your pump is ready to upload


Check that the pump is not on one of these screens.



- No temp basals
- No alarms
- No active bolus
- No low battery
- Verify that Time/Date corresponds to your computer's time and date


THREE EASY STEPS UPLOAD YOUR PUMP OR METER DATA

STEP 1

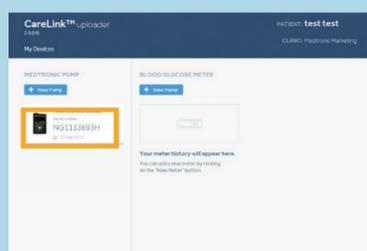

1 Select your device

STEP 3

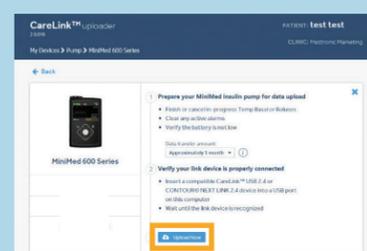

STEP 2

GET EVERYTHING READY
Plug your USB or meter into your computer

- Place your pump as close as possible into your computer
- Keep the meter linked with the pump

- Make sure the device is ready to upload
- Click "Upload Now"

Safety Information: CareLink™ software

CareLink software is intended for use as an adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself.

*Products that appear on this web site may not all be approved in your country. Please contact your local affiliate for further information.

© 2018 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. Third party brands are trademarks of their respective owners.

Medtronic

Medtronic Canada
99 Hereford Street
Brampton, ON
L6Y 0R3

Tel: 1.800.284.4416
diabetesps.canada@medtronic.com
UC201903511 EC

SPEAK WITH YOUR HEALTH CARE PROFESSIONAL



Speak with your Health Care Professional about the CareLink™ system for more efficient data sharing